# hialix®

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# hialix®

Medical device.  
24 lozenges

moisturizing pastilles for throat and larynx  ENT12

with hyaluronic acid and Icelandic moss extract

Carefully read the instructions for use contained in the package, as it contains important information for the patient about the use of **HIALIX®**, and then store it in case the need to re-read.

## 1. What is HIALIX® ?

**HIALIX®** is a medical device in form of lozenges with blackcurrant flavor, which acts melting in the oral cavity, creating a protective film, which protects the mucosa of the oral cavity, throat, vocal cords and larynx from the irritating influence of external factors like bacteria or viruses.

It contains natural extracts of Icelandic moss, mallow and linden, and it is rich in mucinous substances, which coat both the oral cavity and throat protecting and blocking sensory nerve endings (which are responsible for the feeling of sore throat and cough reaction).

**HIALIX®** contains hyaluronic acid (component of the created film), which intensively hydrates dry and irritated mucosa providing relief. Thanks to the properties of its ingredients, **HIALIX®** protects the throat restoring the balance of the irritated and dry mucosa. It also soothes afflictions related to dry cough. This medical device is safe both for adults and children over 6 years of age.

## 2. What is HIALIX® used for ?

**HIALIX®** is used for proper protection and moisturizing the mucosa of the oral cavity, throat, vocal cords and larynx. It acts by coating and moisturizing the mucosa of the oral cavity, throat, vocal cords and larynx, alleviating the feeling of dryness, itching and sore throat, especially when swallowing. It also soothes the symptoms of dry cough.

### It is recommended to use HIALIX®:

- As a support for the inflammation of the oral cavity and throat caused by different factors, e.g. pharyngitis, tonsillitis or aphthosis.
- To ease the symptoms of dry cough and hoarseness.
- In people who intensively overburden the vocal cords (teachers, lecturers, singers, broadcasters, etc.)
- In people with dryness of the oral cavity,
- In people at risk of external factors like dry air, air-conditioned, cold air, contaminated air

## 3. Dosage and how to use

**HIALIX®** should be used according to instructions for use.

If you are not sure or if symptoms persist for more than 7 days, it is recommended to contact your physician or pharmacist.

Adults and children over 6 years of age: 1 or 2 lozenges several times a day. Do not use longer than 30 consecutive days.

Do not use the product after expiration date located on the packaging and when the packaging (blister) is damaged.

Lozenges should not be swallowed or chewed up. It is recommended to leave the lozenge in the mouth until it melts completely.

### Warning:

The medical device contains sweetener: acesulfame K, which is the source of phenylalanine. Excessive consumption can cause laxative effect. Due to the form of the medical device - lozenges - and the possibility of choking it is recommended to use the device only for children who can suck lozenges and on supervision of adult.

## 4. Contraindications:

Hypersensitivity (allergy) to any product ingredient. Diagnosed phenylketonuria.

## 5. How to store HIALIX®:

Store in the original packaging at 15-25°C.

Keep out of reach and sight of children. Do not use after expiration date located on the packaging.

Do not use in case of direct packaging damage (blister).

## 6. Composition:

Extract of Iceland moss (*Cetraria islandica*), extract of high mallow (*Malva sylvestris* L.), extract of linden (*Tilia platyphyllos/Tilia cordata*), sodium hyaluronate, L-ascorbic acid (vitamin C), gum arabic, maltitol syrup, sorbitol syrup, purified water, blackcurrant flavor, coloring anthocyanes, citric acid, acesulfame K, glazing oil with beeswax.

## 7. Packaging description:

**HIALIX®** is packed in blisters containing 12 lozenges. Each unit box contains 2 blisters and the instructions for use

Last revision date: November 2020

Explanation of the symbols presented on the packaging:


Expiry date

LOT

Batch number

Read instructions for use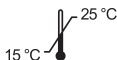

Storage temperature

Do not use if the package is damaged

Manufacturer

REF

Catalogue number

1434

HLX2/II/11/2020  
HIA 24/3.7

Gofarm Sp. z o.o. Sp.k.  
Górników street 21/26  
30-819 Cracow, Polonia  
www.gofarm.pl

**NORMON**

OTC